Carospeed Menox

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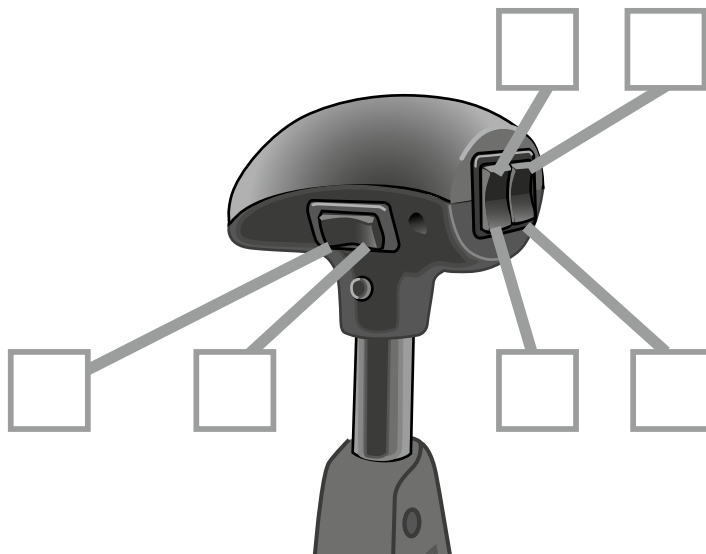
Thank you for choosing a

Carospeed Menox from Autoadapt!

Thank you for choosing us as the supplier of your car adaptation. This product is designed to make it easier for you to drive your car, and we have focused particularly on safety, quality and functionality. The following manual is an integral and important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please feel free to contact your installation engineer. If you wish to share your opinions about the product or the company with us, you can register them on www.autoadapt.com/register. That way, you will be helping us to become a better supplier.

Once again, thank you for placing your confidence in our products!

Autoadapt

Safe vehicle adaptation solutions

For your safety Autoadapt products are designed and tested according to current directives and standards.

Safety

14 Safety measures

Operation

15 Hill-holder

16 Start and drive off

16 Park the car

17 Cruise control

18 Option, primary electrical functions

“Declaration of conformity” at the end of the manual.

Safety

Safety measures

Carospeed Menox may only be used by individuals who has a prescription to do so.

Do not use Carospeed Menox as a handle to get inside the car.

Contact an authorized technician if the Carospeed Menox appears to be wiggly sideways.

Carospeed Menox shall be inspected by an authorized technician if a car crash has occurred.

Operation

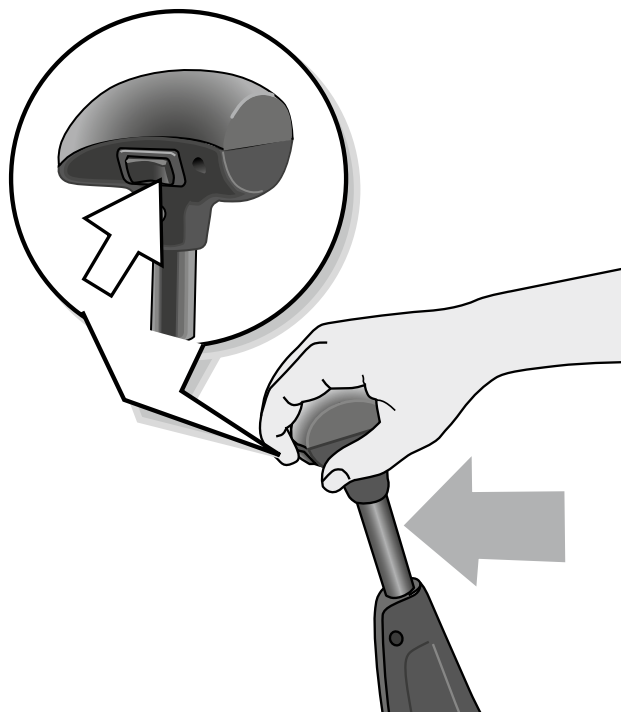
Hill-holder

The hill-holder function enables the user to brake the car at the same time as the gear shift is set. Activate the hill-holder by pressing the button on the back of the handle. It can only be activated while braking the car; when sufficient brake power is achieved and ignition is on.


The hill-holder lets go automatically as soon as the handle is pushed forward (braking).


Never leave the car while the hill-holder is activated.

Start and drive off

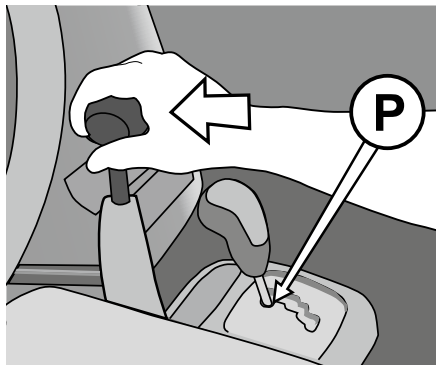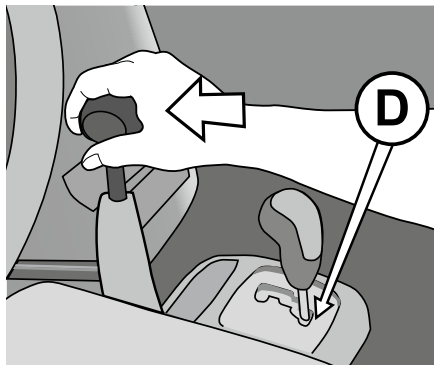
- 1 Start the engine with the gear shift in parking position. Activate the hill-holder. Now the gear shift D or R can be set without the car moving.

- 2 Release the parking brake. Un-activate the hill-holder (press the handle forward and the hill-holder lets go automatically). Now you can start driving.

To brake push the handle forward.
To accelerate pull the handle backwards.

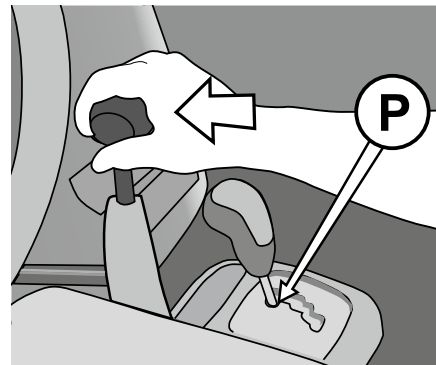Park the car

- 1 Brake the car and activate the hill-holder as soon as the car has stopped. Set the gear shift to parking position (P) and pull the parkingbrake. Un-activate the hill-holder by pushing the handle forward.

The hill-holder shall never be used as substitute for the parking brake.

Cruise control

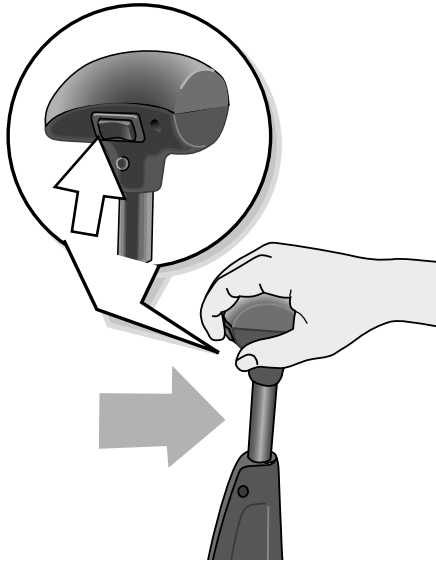
Do not use cruise control in rain or in otherwise slippery conditions.

Do not leave the mechanical cruise control activated when turning off the engine.

EN

Operation

Activating the vehicle's own cruise control

The Cruise control is activated by the same switch as the hill-holder, but can only be activated while accelerating. Accelerate to the desired speed. Press the cruise control button once. Now you can let go of the handle and the speed remains the same.

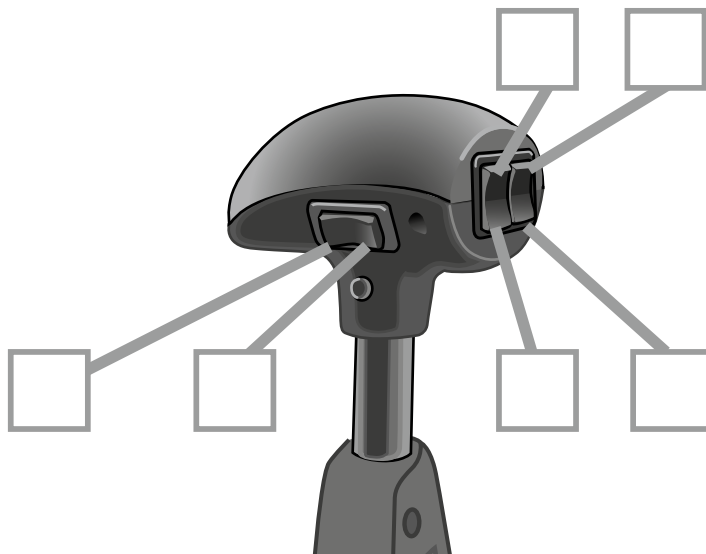
Activating the optional Mechanical Cruise control, for vehicles that lack cruise control function

The mechanical Cruise control is activated by the same switch as the hill-holder, but can only be activated while accelerating. The mechanical cruise control is activated as described in "Activating the vehicle's own cruise control" above. To release just move the handle in any direction.

Option, primary electrical functions

Connections are made according to individual customer needs. The installer is responsible for marking the connections.

- 1. Wiper 
- 2. Wash and wipe 
- 3. Flashers, left/right 
- 4. Headlight 
- 5. Horn 
- 6. Hill-holder 
- 7. Cruise 
- 8. Tiptronic +
- 9. Tiptronic -

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Phone: +46 302 254 00
E-mail: info@autoadapt.com
www.autoadapt.com

Declares that the product /

Carospeed Menox, model Left / Right
hand

Conforms to following directives, standards and regulations /

2006/42/EEC
2007/46 EEC
ECE R-10
ECE R- 21
ISO 3795
ISO/TC22 N 2072
AS 3954.1-1991
AS 3954.2-1991

Autoadapt AB, Stenkullen 2016-03-03

A handwritten signature in black ink, reading "Håkan Sandberg". The signature is stylized with a long horizontal line extending from the end of the name.

Håkan Sandberg
CEO

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005
Quality system certified in accordance with ISO 9001:2008

Illustrations, descriptions and specifications in the user manual are based on current product information.
Autoadapt reserves the right to make alterations without previous notice.
© 2016 Autoadapt AB

www.autoadapt.com

facebook.com/autoadapt
twitter.com/autoadapt

AUTOADAPT AB

Åkerivägen 7
S-443 61 Stenkullen
Sweden
Phone: +46 302 254 00
E-mail: info@autoadapt.com

Worldwide Safe Mobility Solutions

Included in our corporate group are Autoadapt,
BraunAbility, Bruno, Unwin and Foca-Braun.