CAROLIFT

6000/6900

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NL

USER MANUAL

GB

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F

Carolift 6000

Carolift 6900

auto
ADAPT
for your independence

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NL

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
The contents of this instruction must be read and applied before the product is used. Store the User Manual where it can be easily found if necessary. Carolift is a hoist designed for loading and unloading of wheelchairs and scooters in and out of vehicles. Autoadapt AB relinquishes all liability if the Carolift is modified or rebuilt or used for purposes other than those for which it has been designed.

The installation and repair of Carolift must, for reasons of safety and for the warranty to apply, be carried out by Autoadapt AB or by an Autoadapt AB accredited workshop.

1. PRESENTATION

Carolift 6000 and 6900 are hoists with an offset lifting column, which makes it possible to load and unload a wheelchair or scooter positioned to the side of the car's rear section. The advantage, apart from the wheelchair can remain on the pavement when loading, is that the lifter can be installed in vehicles with a sloping rear door. The hoists are operated via a hand control and features electrical functions for lifting, lowering and rotating in and out of the car. In addition, Carolift 6900 has an electrically driven telescopic outer lift arm, so that the load can be guided through narrow door openings.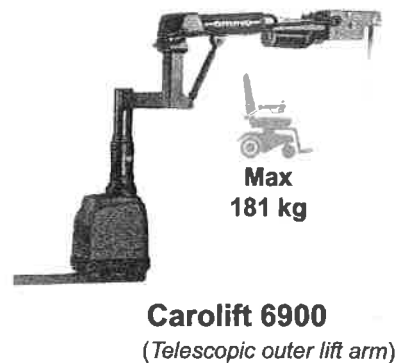


A power switch is located on the lower part of the lifter. This should be switched on in order to use the hoist.

2. GENERAL SAFETY REGULATIONS

-  Keep hands, fingers and feet away from moving components during lift operation.
-  The wheelchair must be securely anchored in the car's boot while being transported.
-  When the wheelchair is to be lifted from the ground the hoist's wire should be vertical as possible, to prevent the wheelchair from swinging.
-  Secure the hand control in the intended place when not in use, this will prevent the unintentional activation of the hoist by loose baggage.
-  Vehicle engine must be shut off, with transmission set in park (parking brake must be engaged on models with manual transmission).
-  Lifting device must be lowered to the floor and secured when driving without a loaded mobility device.

3. LOADING THE WHEELCHAIR Carolift 6000:


The word *scooter* is used in the following text and consequently refers to both a scooter and wheelchair.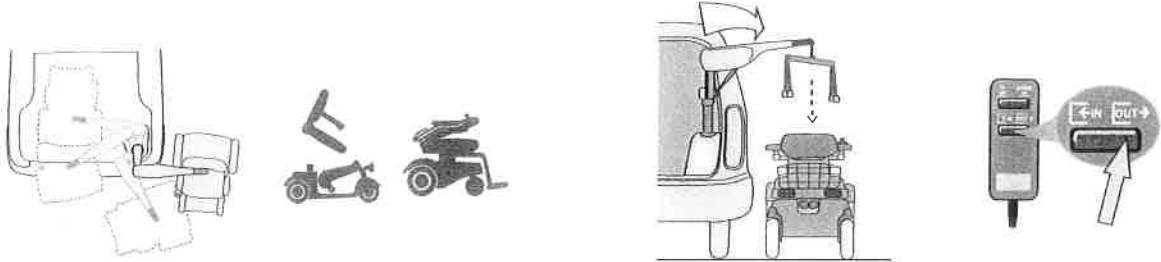

1. Place the wheelchair with the front in the opposite direction compared to the car.

Prepare the wheelchair for loading by removing /folding down the seat back if fitted.

2. Swivel out the hoist using the hand control. Make sure the lift belt with lifting device is positioned as directly above the wheelchair's fixing point as possible. Otherwise there is a risk that the wheelchair can start to swing when lifted.

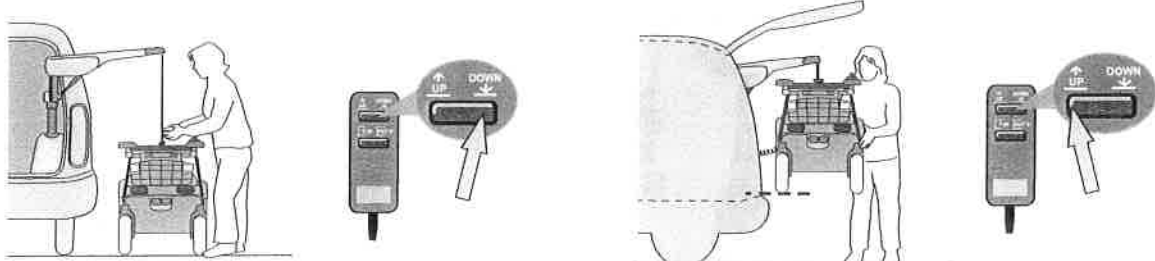3. Lower the lift belt with lifting device, using the hand control. Hook/tension the lifting device in the wheelchair's attachment.

Check that the wheelchair is secured correctly in the hoist!

4. Lift the wheelchair using the hand control. At the same time grip the wheelchair, guide it up.

Lift the wheelchair **just above** the level of the door opening's lower edge.

5. Rotate the wheelchair using the hand control. At the same time guide the wheelchair with one hand, so that it comes in the right position in the car.
6. Lower so that the load stands firmly on the floor of the car. **Lock the brakes on the wheelchair!**

7. The wheelchair should be completely lowered onto the car floor so that it stands firmly! The lift belt can remain hooked in the wheelchair, but in the tensioned position, so that it does not come loose during transport. Hang the hand control in its holder on the hoist to prevent unintentional activation.


Secure the wheelchair in the car, using the appropriate anchoring devices!

4. LOADING THE WHEELCHAIR

Carolift 6900:

The word *scooter* is used in the following text and consequently refers to both a scooter and wheelchair.

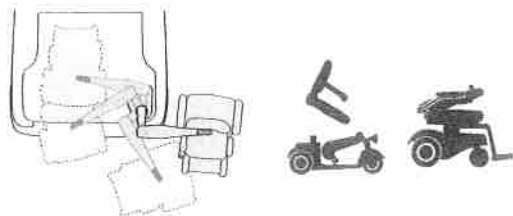1. Place the wheelchair with the front in the opposite direction compared to the car.

Prepare the wheelchair for loading by removing /folding down the seat back if fitted.


2. Swivel out the hoist using the hand control.

Extend the outer lift arm using the **EXTEND** button. Make sure the lift belt with lifting device is positioned as directly above the wheelchair's attachment as possible. Otherwise there is a risk that the wheelchair can start swinging when lifted.

3. Lower the lift belt with lifting device, using the hand control. Hook/tension the lifting device in the wheelchair's attachment.

Check that the wheelchair is secured correctly in the hoist!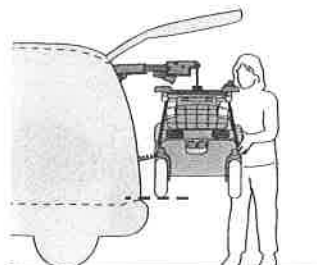

4. Lift the wheelchair using the hand control. At the same time grip the wheelchair, guide it up. Lift the wheelchair **just above** the level of the door opening's lower edge.

5. When the wheelchair is above the level of the door opening's lower edge the hoist's lift arm should be retracted. This is done via the hand control's **RETRACT** button. Rotate the wheelchair using the hand controller. At the same time guide the wheelchair with one hand, so that it comes in the right position in the car.

6. Lower the wheelchair and **lock the brakes**.

7. The wheelchair should be **completely lowered onto the car floor so that it stands firmly!** The lift belt can remain hooked in the wheelchair, but in the tensioned position, so that it does not come loose during transport.


Hang the hand control in its holder to unintentional operation.

Secure the wheelchair, using the appropriate anchoring devices!

5. UNLOADING THE WHEELCHAIR

Carolift 6000:

The word scooter is used in the following text and consequently refers to both a scooter and wheelchair.

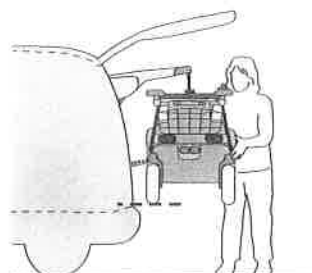1. Loosen the wheelchair's floor anchoring.
Lift up the wheelchair, using the hand control, so that it can pass freely over the lower edge of the door opening.
2. Turn the wheelchair to a suitable position to the side of the car's rear section.

⚠ Ensure that no persons, animals or objects are within the loading zone!

3. Lower the wheelchair, using the hand control. Loosen the lifting device from the wheelchair's attachment.
4. Swivel the hoist into the car using the hand control.

Wind in the lift belt and lifting device, using the hand control.

⚠ Place the hand control in the holder on the hoist, to prevent unintentional operation.

⚠ IMPORTANT: Ensure the lifting device is secured well, so that it cannot injure passengers in the event of an accident or when braking heavily.

6. UNLOADING THE WHEELCHAIR Carolift 6900


The word scooter is used in the following text and consequently refers to both a scooter and wheelchair.

1. Make sure that the outer lift arm is fully retracted.
Loosen the wheelchair's floor anchoring. Lift up the wheelchair, using the hand control, so that it can pass freely over the lower edge of the door opening.

! Ensure that persons, animals or objects are not within loading zone!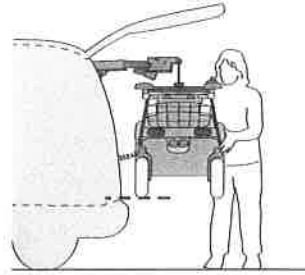


2. Swivel the wheelchair out through the door opening. When the wheelchair is outside of the door opening, extend the outer lift arm using the **EXTEND** button on the hand control.

Turn the wheelchair to a suitable position to the side of the car's rear section.

3. Lower the wheelchair, using the hand control. Loosen the lifting device from the wheelchair's attachment.

Wind in the lift belt and lifting device, using the hand control.

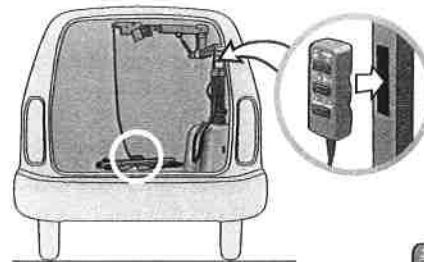4. Swivel the lifter into the car using the hand control.

! Place the hand control in the holder on the hoist, so that the lifter cannot be activated unintentionally.

! **IMPORTANT:** Ensure the lifting device is secured well, so that it cannot injure passengers in the event of an accident or when braking heavily.

7. FOLDABLE HOIST HEAD (Carolift 6000)

The Hoist head of Carolift 6000 is foldable.

Remove the pin placed at the back of the hoist head.

Fold down the hoist head.

8. EMERGENCY BACKUP OPERATION

The Carolift is equipped with a hand crank to serve in case of a power failure.

1. Remove the 4 screws securing the cover to the hoist base.

2. Compress the boot with your hand and tuck it inside the cover.


3. Lift the cover up and rotate it to provide access to the crank placed on the base motor. Slide the crank out from behind the wire tie. Do not cut the tie.

4. Place the crank over the motor shaft.

5. Rotate the crank until the lift has returned to its end position.

6. Slide the crank back behind the wire tie.

7. Have the lift examined by an authorized Autoadapt dealer.

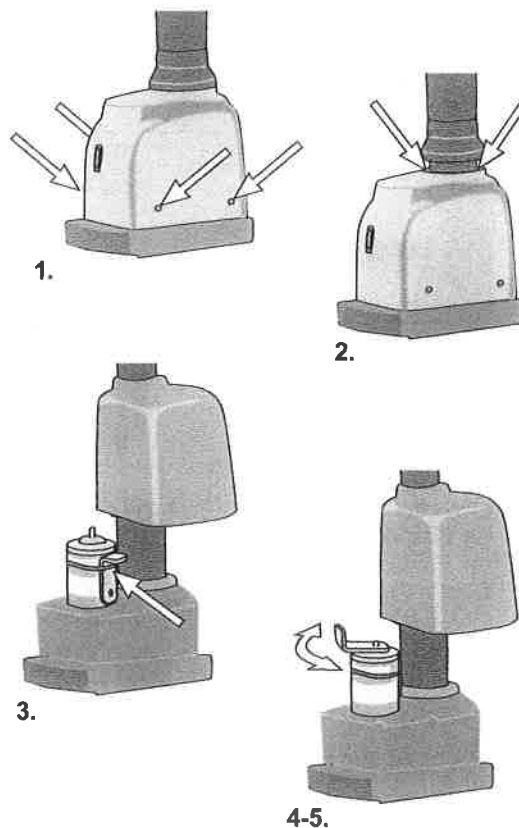9. FAULT TRACING

Symptom	Cause	Action
<i>The lifter does not work when the control buttons are pressed in.</i>	<i>The switch on the hand control is in the "off" position.</i>	<i>Press the switch button so that it is switched on.</i> 
<i>When activated upwards the lifting belt runs downwards instead, and when activated downwards it runs upwards.</i>	<i>The lifter has been run down too far and the lifting strop has twisted in the opposite direction around the reel.</i>	<i>Run the lifter down as far as possible and then up again until the strop winds the right way.</i>
<i>Abnormal noise from the motor when operating the lifter.</i>	<i>Defective motor or worn pinion.</i>	<i>Replace the motor.</i>

10. MAINTENANCE

All maintenance must be carried out by one of Autoadapt's accredited dealers.

The Carolift lifters should be inspected at least once a year to check for signs of wear or defective components and to carry out preventive maintenance.

- Inspection of the lifting strop
- Inspection of electrical components
- Inspection and adjustment of the lifting device

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